# Gel Compress Cold & Hot Treatment

## ❄️ COLD/HOT GEL COMPRESS Instructions for use

### AREAS OF APPLICATION:

Cold/hot compresses are for external use only by untrained non-specialists both in medical institutions and in domestic environments. Cold/hot compresses can be used in the following areas of application:

**Cold treatment:** In cases of bruises, inflammation, insect bites, hangovers, headache, minor burns, migraine, nose bleeds, contusions, swelling, dislocation, sprains, tennis elbow, toothache, strained muscles.

**Hot treatment:** In cases of lumbago, menstruation pain, aching muscles, rheumatic pain, backache, colds, chills, tension.

### DO NOT USE IN CASE OF:

Circulatory disorder and cryaesthesia, Raynaud syndrome. Patients suffering from cardiovascular disease and diabetes mellitus may only use the product following consultation with their doctor.

### APPLICATION INFORMATION:

**Use as a cold compress:** Cool the compress in the freezer compartment down to approx. -18 °C for 2 hours before use. Wind a thin cloth around the compress for use. We recommend keeping the compress stored in the freezer compartment after use so that it is always ready for use.

**Use as a hot compress:** Before use place the compress in hot water up to +80 °C for approx. 4 minutes or in the microwave at 280 W for approx. 2 minutes (under constant observation). After the heating-up process, remove the compress and knead it well to distribute the heat. Wind a thin cloth around the compress for use.

⚠️ **Caution!** Overheating can destroy the compresses. Leaking or emptied compresses must not be heated and used again.

### NOTE:

The gel used in the compress is not toxic. Should any gel get onto the skin or into your eyes, rinse off or out using plenty of water. Consult a doctor if necessary. There have been no reports of irritation of the skin and mucous membrane.

Keep the cold/hot compress in a cool and dry place protected from sunlight. After application the cold/hot compress can be cleaned with standard washing-up liquid and water if necessary. Damaged, leaking compresses must not be used again and must be disposed of with household waste. Keep cold/hot compress out of the reach of children.

Please report any serious incidents which happen in connection with the product to Gello GmbH Geltechnik and the authorities responsible.

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Apie visus rimtus incidentus, susijusius su gaminiu, praneškite „Gello GmbH Geltechnik“ ir atsakingoms institucijoms.
PVG Pharma GmbH,